



Applying Enluxtra Dressing for NPWT



For Authorized Healthcare Professionals Only

Applying Enluxtra Dressing

- To achieve a proper fit
 - Make sure the dressing size will overlap HEALTHY skin by at least 1" on all sides
- Apply the dressing with the PRINTED SIDE OUT
 - Ensure full contact with wound bed
- Position wound in the center of the dressing
 - To allow negative pressure to draw exudate up and away from the wound
 - To ensure that gravity doesn't shorten the usable life of the dressing

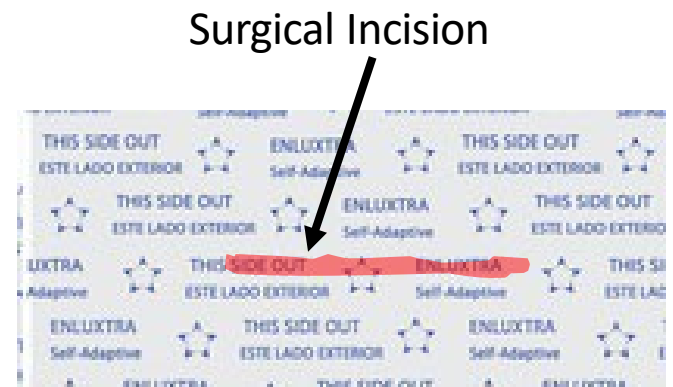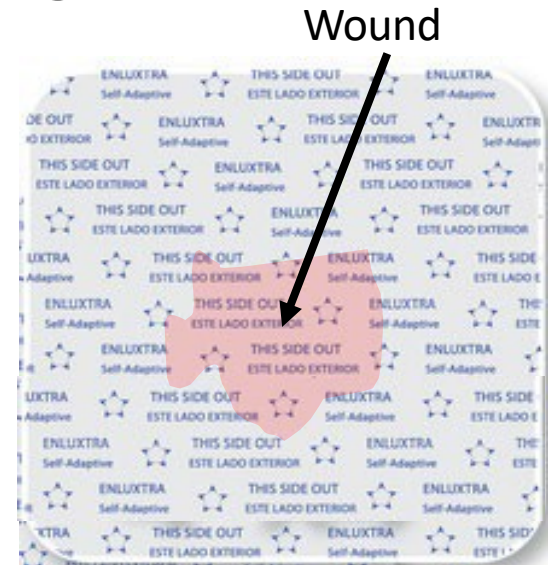


Tips on Enluxtra Application to the Wound

- Allow at least 1” of excess around the wound edges to cover the compromised peri-wound skin
 - Peri-Wound area is fragile and vulnerable to infection
 - Microcracks exist in the peri-wound area making it easier for bacteria to penetrate the skin
- Never cut the Enluxtra to the same size of the wound
 - Electra may be trimmed to for optimal placement or avoiding interference with body parts
 - More dressing material around the peri-wound area allows for more exudate absorption before a dressing change is needed


Apply Outer Drape

- Cut the included outer drape to size that is 2-4 inches larger than the Enluxtra dressing
- Apply the drape over the entire Enluxtra dressing allowing 1 to 2 inches around each side of the Enluxtra

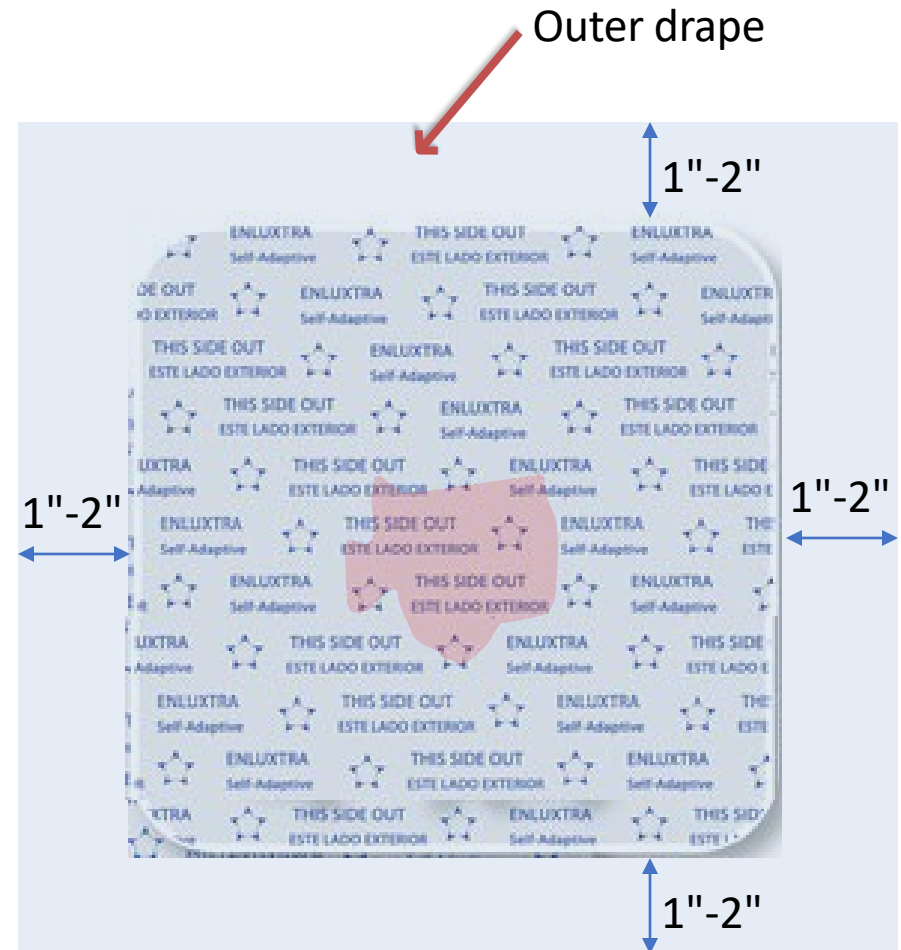

Creating an Opening in the Outer Drape

- In the corner of Enluxtra, choose a point located 1" from Enluxtra edges.
- Pinch the outer drape together with Enluxtra backing film at this point and pull them up slightly.
- Use medical shears to make a cut in the form of cross of 1/2" length through both the outer drape and Enluxtra backing film. It creates an opening for connecting to vacuum tubing.

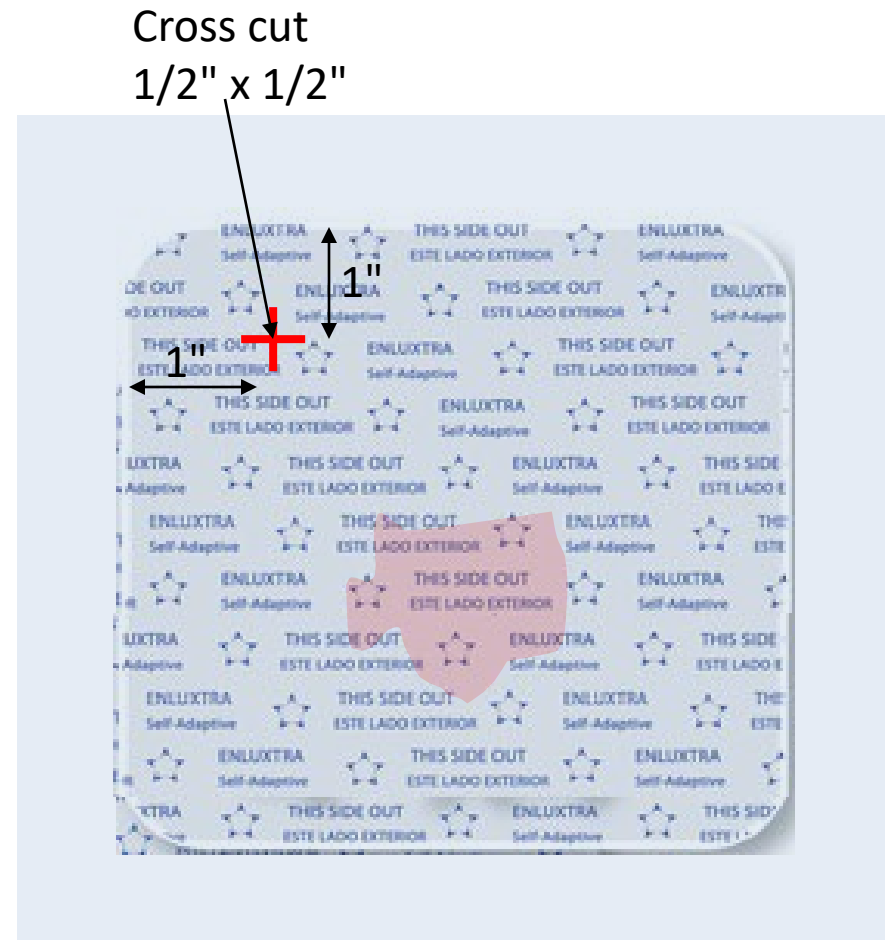

Place Vacuum Port onto Dressing

- *Follow the manufacturer's instructions for placement and connecting the Vacuum Port and tubing*
- Place Vacuum Port over the opening created on both the Enluxtra top layer and outer drape
- Connect the Vacuum Port fitting to the pump tubing fitting



When to Replace a Dressing

- Dressing should be changed after 1 - 3 days depending on the risk of infection or drainage level
- Or if the exudate reaches the Vacuum Port opening or the dressing edge

